

HealthSmart IFU

Item # 510-1085-1900

Instructions for Use (IFU)


3" (7.62cm) W x 6" (15.24cm) H

healthsmart®

Universal Walker Basket

Item 510-1085-1900

Includes: Removable molded plastic tray with cup holder,
hook and loop straps, and instructions for use.
Fits most walkers.

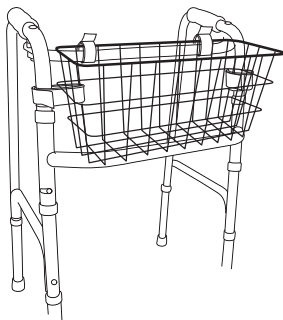

ASSEMBLY INSTRUCTIONS:

1. Place the basket on the walker crossbar.
2. Wrap and secure the two small hook and loop straps around the top of the crossbar and basket.
3. Wrap and secure the two long hook and loop straps around the legs of the walker and outside corners of the basket.

CAUTION: Placing heavy objects inside the basket may cause the walker to become unbalanced and difficult to maneuver. Be sure to distribute weight evenly inside basket and use caution when using walker while items are in the basket.

Manufactured for:

HealthSmart® International

1931 Norman Drive

Waukegan, IL 60085 USA

Register this product at

LiveHealthSmart.com

©2021 HealthSmart® International

#910-1085-1900 05/21

Made in China

1-Year
Limited Warranty